

How to request a service call through VirtualCARE

Exclusively for the Centargo CT Fluid Delivery System.








Select the **Admin Tile** on the Control Room Unit (CRU) or Workflow Hub* and then select the **VirtualCARE tile.**





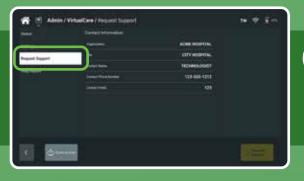



If the site information is missing or incorrect, touch the area you wish to update and enter the correct information.

The CRU/Workflow Hub* will display a waiting for response message, which could take up to 2 minutes.

- > If the request was successful, the CRU or Workflow Hub* displays a service notified message.
- ➤ If the request was not successful, the CRU/Workflow Hub* displays a notification error message. Contact the Technical Assistance Centre at 1-800-633-7237 for further assistance.

^{*}Your Centargo CT Injection Systems may either have the workflow hub or the control room unit (CRU), the interface is the same.



Clear Direction. From Diagnosis to Care.

Bayer reserves the right to modify the specifications and features described herein or to discontinue any product or service identified in this publication at any time without prior notice or obligation. Please contact your authorized Bayer representative

Bayer, the Bayer Cross, and Centargo are trademarks owned by and/or registered to Bayer in the U.S. and/or other countries. Other trademarks and company names mentioned herein are properties of their respective owners and are used herein solely for informational purposes. No relationship or endorsement should be inferred or implied.

Other trademarks and company names mentioned herein are properties of their respective owners and are used herein solely for informational purposes. No relationship or endorsement should be inferred or implied.

© 2025 Bayer Inc. This material may not be reproduced, displayed, modified or distributed without the express prior written consent of Bayer.



Bayer Inc.

2920 Matheson Blvd East Mississauga (Ontario) L4W 5R6 Telephone: 800-268-1432

Fax: 800-567-1710