

The Protector by Your Side

Vial Planner Tool Instructions





Enter the vial assay time (or activity reference time) for the multi-dose vial. The desired time may be typed directly into the data entry field in 24-hours format. If the time separator is omitted, it will be added automatically.



Enter the time that the MEDRAD[®] Intego PET Infusion System Source Administration Set (SAS) will be primed.

Enter the time of administration and the dose for the first patient. Click the + button to add a dose. Click the - button to remove a selected dose.





Repeat Step 3 for each planned patient

Press the Calculate button.



Note: The displayed information is fictitious and does not represent any specific patient or dosing data!



The Protector by Your Side

Vial Planner Tool Instructions







Choose a vial volume and activity within the valid vial configuration graph. Placing the mouse over one of the points in the graph will display the activity and volume values.

